
Sure Ortho Light Band Cement Blue



Instructions for Use

- 1) Clean teeth with plain pumice using a rotating bristle brush or rubber cup. Rinse thoroughly.
- 2) Isolate and air-dry the bonded surfaces. Etching is not required.
- 3) Syringe a thin layer of the cement directly onto the tooth-bearing surfaces of the band.
- 4) Immediately seat band onto tooth. Final band adjustment must be made before the cement begins to cure from the ambient light, approximately 60 seconds.
- 5) Sure Ortho Light Band Cement Blue can be easily wiped away with a cotton roll before it starts setting or removed with a scaler after it has set. The blue color of the uncured cement helps identify access material.
- 7) Light cure for 20 seconds from two opposite directions.
- 8) Keep teeth isolated for 3 min after last bond.
- 9) When bonding to Amalgam, roughen amalgam surface with a diamond bur or with a Micro Etcher prior to applying the Sure Ortho Light Band Cement Blue.

Properties

Easy bonding of bands to tooth surface.

Uses light cure adhesive technology to provide you with additional working time to ensure accurate band placement.

Releases Fluoride.

Contains the adhesive promoter.

4-Meta, for strong reliable chemical bonding with tooth, metal and porcelain surfaces.

Precautions

Refrigeration will increase viscosity, making syringing more difficult, and will retard setting time.

Contains polymerizable monomers which may cause allergic contact dermatitis in susceptible persons. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.

Storage

Do not expose to temperature exceeding 24°C/75°F.

Do not expose to direct sunlight.

Refrigerate for maximum shelf life. Do not freeze.

Recap syringe after each use.

Shelf life is 24 months from date of manufacture.